



**End User Information for Model F-Y3-A KN95 Medical Protective Mask**

Date: 13/06/2020

**Manufacturer:**

- Name: Jiande Chaomei Daily Chemical Co., Ltd
- Address: Yang Cun Qiao Town, Shangshan Industrial Area, Jiande, China
- eMail : [rolen@cmmask.com](mailto:rolen@cmmask.com)
- Website: [www.cmmask.com](http://www.cmmask.com) F-Y3-A: <http://www.cmmask.net/products/details/yfkhz-135.html>

**Product Usage:**

Model F-Y3-A can be used in medical environment. It helps to filter particulate matter in the air and blocks droplets, blood, fluids and secretions, etc.

**How to Fit:**

- Unfold the mask.
- Position the mask on your face so that the nose clip rests on your nose and the bottom of the mask covers your chin.
- Pull the top elastic band over your head, and position it at the neck and below your ears.
- Pull the bottom elastic band over the top of your head and position it around your head above your ears.
- Use both hands to mold the metal nose clip around your nose to achieve a secure seal.
- Place both hands completely over the mask and exhale. If air leaks around the nose, readjust the nose piece. If air leaks at the mask edges, adjust the elastic bands.



**Caution:**

- Use before expiration date.
- The product is disposable and intended to be used only once. DO NOT attempt to re-use after 8 hours.
- The product should be carefully inspected before use. If the packaging is damaged, DO NOT use the respirator.
- It is recommended that the mask not be used if beards, facial hair or other conditions exist (skin rash, acne, etc.) that prevent an effective seal between the face and the mask.
- If the mask was damaged or contaminated during use, or if breathing with the mask becomes increasingly difficult, replace it with a new one.